Ceka med Paraffingaze / Paraffin Gauze Dressing

Beschreibung / Description

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Ceka med Paraffin Gauze Dressing is a sterile, disposable product consisting mainly of Paraffin and gauze. Paraffin is not hydrophilic and cannot be absorbed. Paraffin gauze as a wound covering protects and insulates the wound and prevents leakage of fluid. Paraffin gauze supports wound healing, keeps the skin moist and prevents contact of the wound with airborne bacteria. The dressing does not adhere to the wound when the dressing is changed, effectively protecting the wound from secondary injury.

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Ceka med Paraffin Gauze Dressing is suitable to cover the wound of minor burns, keep the wound moist and help skin epidermis to regenerate.

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1. Do not use on wounds with lots exudates.

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Ceka med Paraffin Gauze Dressing may be used by lay person under supervision of health care professionals.

- 1. Wash the wound with normal saline.*
- 2. After the skin around the wound is dry, select the suitable size of the product, and the product should cover the wound completely.*
- 3. Fix with secondary dressing.*

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1. Paraffin Gauze Dressing can be used after unsealing, but it is not allowed to be reused, in case of cross infection.

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- 1. Do not reuse. Do not use the dressing if the sterilized packaging is damaged or open.*
- 2. This product is single use only and is supplied sterile. Discard any unused portion of the product after dressing the wound.*

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- 1. Keep in a clean and dry place in specified temperature range and away from sunlight.*
- 2. Disposal should be handled according to local environmental procedures.*

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