Ceka med Extra starke Saugkompressen / Super absorbent dressing

Beschreibung / Description

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Ceka med Super absorbent dressing is a sterile dressing for use on high exuding wounds, consisting of an air permeable and waterproof nonwoven layer, a super absorbent pad layer and a hydro-penetrating nonwoven layer. The triple-layered construction facilitates fluid management to provide the optimal moist wound environment which leads to the promotion of maceration and damage to peri-wound skin.

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Ceka med Super absorbent dressing is indicated for management of high exuding wounds, including: leg ulcers, pressure ulcers, diabetic foot ulcers and dehiscd surgical wounds.

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1. Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption. Do not use on third-degree burns.

2. Ceka med Super absorbent dressing should not be used on persons with a known allergy or hypersensitivity to the dressing or its components.

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1. After wound is cleaned and sterilized, the appropriate size dressing is selected to allow for sufficient space to the wound edge.

2. Ceka med Super absorbent dressing can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, Ceka med Super absorbent dressing must be in close contact with the wound bed to ensure wound fluid is absorbed. Secure accordingly with bandage, tapes or compression.

3. The frequency of changing dressing depends on the amount of exudates.

4. Dressing removal can be facilitated by moistening the dressing with saline solution.

5. Dispose of as normal waste. If contaminated, follow local regulations.

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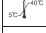
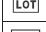
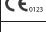
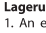
1. Do not reuse. Do not use the dressing if the sterilized packaging is damaged or open.

2. Do not cut the dressing, the cover must remain intact.

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1. Keep in a clean and dry place in specified temperature range and away from sunlight.

2. Disposal should be handled according to local environmental procedures.

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3 years from date of manufacture